

AL6000-Pro Professional Breathalyser

State-of-the-art technology in a compact durable unit, combined with quality and simplicity

The AL-6000 Pro Digital Breathalyser (Alcohol Tester / Detector)

The AL6000 Pro has been tested and certified to **Australian Standard AS3547** as a Type II hand held device. It is the first semiconductor breathalyser with Australian Standard tested and certified.

The AL6000 Pro is accurate enough for police law enforcement use as well as cost effective for personal use.

The AL6000 Pro is the world's first and only breathalyser that can be operated in dual-mode; it can be operated in a normal mode to measure specific blood alcohol concentration, or it can be used as an alcohol responder (share between multiple people) to test the presence of alcohol. AL-6000 Pro calibrating with high accuracy with ranges of 0.00 to 0.40%BAC.

Product Quality :

- Australian Standard AS3547 CERTIFIED
- Registered CE Approval.
- Registered DOT Approval.
- FDA Approval.
- Product Made in Korea .

Category Contents

- Indication of B.A.C Normal Mode : 0.00 to 0.40% B.A.C by LED display.
- Fast Mode : Zero, Low, High 3 steps
- Warning Up : Within 20 seconds or less
- Response time : Within 5 seconds Normal Mode, 3 Seconds Fast Mode
- Recycle time : 10 seconds
- Mouth piece : Extra mouthpieces available from SKMC
- Sensor : Semiconductor type alcohol sensor(Gas flow sensor built-in)
- Continuous using time without battery recharge : over 200 times
- Power supply : two 1.5V "AA" Alkaline batteries
- Weight : 105g (Including batteries)
- Dimension : (mm) 137 X 73 X 44
- Remark : Auto power off , Battery Low indication

Warranty: 1 Year

AL-6000 Pro User's Guide

Press the Power Switch to turn the unit on. The cumulative test count will be displayed on the screen.

Press the Power Switch one more time and you will see the letters 'NOR' in the display, which indicates Normal Mode operation. (Press again for FAST mode) There will be an automatic countdown as the unit warms up.

Secure your lips around the mouthpiece (or blow directly without a mouthpiece). Blow sharply and steadily into the unit for about 5 seconds. You will hear two beeps when the device has finished taking the breath sample. You will see the digit moving left to right in the display as the device is analysing the sample.

After analysis is complete, you will see the blood alcohol concentration (BAC) results displayed.

Only \$165!



Bonus Carry-Case & extra mouth pieces included!

Promotional Offer on during May! Only \$165! (RRP \$195)

Know your limit and call us today on (03) 9416-4666 or visit stkildamarinaclub.com